



VS216 5 Litre Commode Pot

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the 5 Litre Commode Pot from Aidapt.

NB. A RISK ASSESSMENT SHOULD BE UNDERTAKEN AS TO THE SUITABILITY OF THE PRODUCT FOR THE PARTICULAR USAGE.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage if you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

FITTING & REMOVING COMMUNE POTS (ALL COMMUNES)

To remove the commode pot, simply lift the toilet seat and fix the lid provided onto the bucket to avoid splashing.

Carefully raise the bucket and dispose of the contents into a proper sluice or WC.

Never use an open drain, sink or basin; this could cause a serious health risk.

Clean out the residue with a suitable disinfectant and refit to the commode after use.

CLEANING

Clean your Commode Pot using a non-abrasive cleaner. Abrasive cleaning pads may damage the product beyond repair and should not be used. The Commode Pot can be cleaned with a pressure washer.

When using moist heat the equipment can be held at 81°C for at least one minute to achieve disinfection.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety. **If in any doubt, please do not issue or use, immediately contact your supplier for service support.**

IMPORTANT INFORMATION

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.com Email: sales@aidapt.co.uk