do more  
taskaprosthetics.com

 **TASKA™**

## TASKA Gen1 Power System User Guide

Strømsystem Brugervejledning  
Stromversorgungssystem Anleitung für Anwender  
Manual del usuario del sistema de alimentación  
Virtajärjestelmän käyttöopas  
Système d'alimentation Guide de l'utilisateur  
Notendahandbók fyrir rafmagnskerfi  
Sistema d'alimentazione - Guida per l'utente  
Strømsystem Brukerveiledning  
Guia do utilizador do sistema de alimentação  
Руководство по системе питания  
для пользователя  
Používateľská príručka k napájacíemu systému  
Strömförsörjningssystem Användarhandbok

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**IS**

A Hleðsluinnstungat  
B Aflihnappur

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**EN****DA****DE****ES****FI****FR****IS****IT****NO****PT****RU****SK****SV**

EN

The power button turns the TASKA Hand (or other terminal device) on or off.

### To charge your TASKA Battery

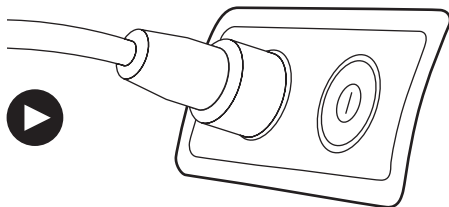
1. Remove prosthesis from your residual limb.
2. Connect charger to power source. See specifications for charger operating limits.
3. Connect the magnetic tip of the charger's lead to the magnetic charging socket on the Power Switch (see figure 1).
4. The light on the mains or car charger will change from green to red to show that charging is in progress.

5. When charging is complete, the light on the charger will return to green.

### Battery care

When well maintained, your TASKA Battery will offer excellent performance. To achieve the best results we recommend:

- charge the battery every night
- charge the battery before storing for long periods
- do not let the battery become completely flat
- replace the battery every 12 months.

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EN

**Battery warnings**

- Do not charge your battery outside of the environmental operating conditions specified.
- If your battery is leaking or you notice fluid in your socket, contact your clinician immediately for repair and cleaning.
- Do not expose the battery to naked flames.
- Do not bend, pierce, or damage the case of the battery.
- If you notice that the battery is swollen or unusually hot immediately stop using or charging your prostheses and contact your clinician.
- Do not expose to temperatures below -10°C (14°F) or above 60°C (140°F).

**Power System warnings**

- Your Power System must be installed by a trained and certified clinician.
- TASKA Prosthetics is not responsible for the waterproofness of the prosthetic socket or Power System. Please discuss waterproof levels with your clinician.
- Contact your clinician if something is wrong with your Power System.
- If your Power System is broken or shows visible signs of damage, please contact your prosthetist for repair options.
- Do not try to repair or modify the Power System yourself. Repairs or modifications by anyone not authorised by TASKA Prosthetics will invalidate your warranty.

- Only use a TASKA Charger and Power Switch to charge TASKA Batteries.
- Do not wear your prosthesis (socket) while charging TASKA Batteries.
- Do not charge TASKA Batteries in the presence of flammable liquids or gas.
- Dispose of batteries and other components according to your local regulations.

**Specifications**

|  |
|--|
| Operating voltage: 7.4V nominal          |
| Maximum current: 5.5A                    |
| Battery weight: 88.6g                    |
| Mains charger input voltage: 100-240V AC |
| Mains charger input frequency: 50/60Hz   |
| Car charger voltage: 10-30V DC           |

**TASKA Power System environmental conditions**

|  |
|--|
| Charging temperature: 0°C to 45°C (32°F to 113°F)                                |
| Operating temperature range: -10°C to 45°C (14°F to 113°F)                       |
| Storage temperature range: -10°C to 35°C (14°F to 95°F)                          |
| Operating and storage humidity range: 5 to 90% relative humidity, non-condensing |

**Compliance and compatibility**

Visit [taskaprosthetics.com](http://taskaprosthetics.com) for up-to-date compliance and compatibility information.

If a serious incident occurs in relation to this device it should be reported to TASKA Prosthetics and the competent

authority of the country you live in. For further information on the TASKA Power system, Contact your clinician.

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